



**Operator's
Manual**



FreeStyle



Optium Neo **H**

Blood Glucose and Ketone Monitoring System



Symbols

These symbols are associated with your meter.

















	Consult instructions for use		Use-by date
	Temperature limit		<i>In vitro</i> diagnostic medical device
	Manufacturer		Catalogue number
	CE Mark		Recycle
	Batch code		Serial number
	Date of manufacture		Biological risks
	The European Battery Directive requires separate collection of spent batteries, aiming to facilitate recycling and to protect the environment. The batteries in this product should be removed and disposed in accordance with local regulations for separate collection of spent batteries.		Sterilized using irradiation (lancets only)
			Do not re-use
			Do not drink

Table of Contents

Intended Use	1	Understanding Blood Glucose Test Results	20
FreeStyle Optium Neo H At-A-Glance	2	Low Blood Glucose Results	21
Meter Symbols	3	High Blood Glucose Results	22
Using the Meter	4	Understanding Blood Ketone Test Results	23
Turning the Meter On and Off	4	View Logbook	25
Checking the Meter Screen Every Time You Turn It On	4	Viewing Logbook Events	25
Setting Up the Meter	5	Viewing Blood Glucose Averages	26
Set Time	5	Glucose and Ketone Control Solution Tests	28
Set Date	7	Performing a Control Solution Test	29
Enabling Out-of-Range Indicators and Quality Control Reminder	8	Transferring Meter Data to a Computer	33
Introduction	8	Error Messages	34
Preparing the Meter for Glucose and Ketone Testing	12	Maintenance	37
Calibrating the Meter for Glucose and Ketone Testing	12	Replacing the Batteries	37
Preparing the Meter	13	Cleaning the Meter	39
Confirm the LOT Number	14	Troubleshooting	40
Testing Blood Glucose or Blood Ketone	15	Meter Specifications	42
Test Site	15	References	44
Performing a Blood Glucose or Blood Ketone Test	16	Warranty Statement	45

Intended Use

The FreeStyle Optium Neo H Blood Glucose and Ketone Monitoring System is for professional use to measure glucose and ketone (β -hydroxybutyrate) in fresh whole blood outside the body (*in vitro* diagnostic use).

IMPORTANT:

- Use only **FreeStyle Optium Neo H** blood glucose test strips and **FreeStyle Optium H** blood β -Ketone test strips. Other test strips may produce inaccurate results.

Potential infection risk: Healthcare professionals performing blood tests with this system on multiple patients must always wear gloves and should follow the infection control policies and procedures approved by their facility.

- See test strip instructions for use for more information about sample types.
- Read the instructions in this Operator's Manual. Failure to follow instructions may cause incorrect results. Practice the testing procedures before using the meter.
- Observe caution when using around children. Small parts may constitute a choking hazard.

How FreeStyle Optium Neo H Blood Glucose Monitoring System Works

When you insert a test strip into the meter, the sample drop symbol shows on the meter's display window. When a blood sample or control solution sample is applied to the test strip, the glucose or ketone reacts with the chemicals on the test strip. This reaction produces a small electrical current that is measured. The result shows on the meter's display window.

FreeStyle Optium Neo H At-A-Glance

Display Window

Displays meter Home screen (shown here)
Displays test results and other important information

Logbook

Up Button

Mark a venous blood test
Adjust settings

USB Port

Insert a data cable here for uploading test results to a computer (data management system required)

Power Button

Turn meter on and off
Return to the Home screen
Save settings

Down Button

Mark a control solution test
Adjust settings

Strip Port


Insert a test strip here

Target Area







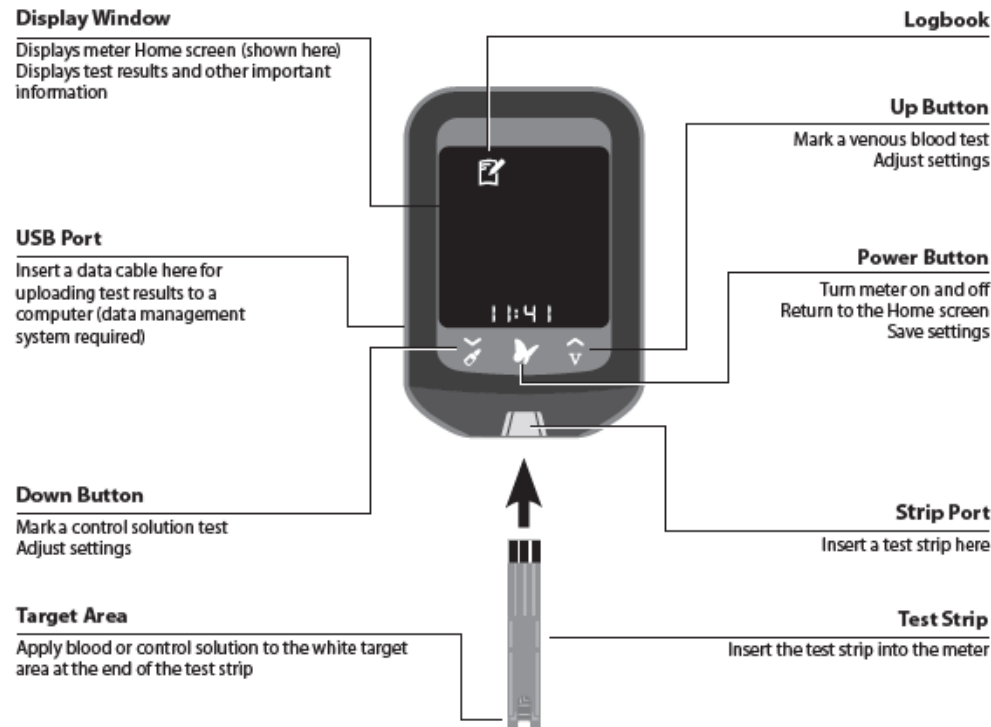








Apply blood or control solution to the white target area at the end of the test strip

Test Strip

Insert the test strip into the meter



Meter Symbols


Symbol	What It Means	Symbol	What It Means
	Logbook		Control solution mark
	Meter ready for sample application		Quality Control test is due
	Control solution result		Number of days
	Ketone		Control solution test
	Connected to computer		Low battery
	High Out-of-Range indicator		Venous blood mark
	Low Out-of-Range indicator		Venous blood test
	Setup mode		

Using the Meter

Turning the Meter On and Off


To turn the meter on:	To turn the meter off:
<ul style="list-style-type: none"> • Press , or • Insert a strip 	<ul style="list-style-type: none"> • Press and hold  for 3 seconds, or • Do nothing for 2 minutes

Checking the Meter Screen Every Time You Turn It On

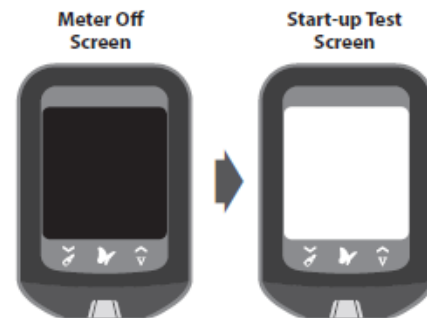

The meter screen should be fully black when powered off. Each time you turn on the meter, a white start-up test screen will appear for 1 second.





If you see any *white* segments in the black off screen, or any *black* segments in the white test screen, there may be a problem with the meter. Contact Customer Services.



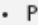











Note: If the meter battery is low,  will appear in both the meter off screen and start-up test screen.

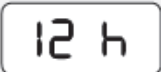Setting Up the Meter



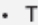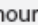


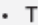
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



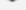

To confirm that the date and time are set correctly, follow the steps below.

Set Time

Step	Action
1	 <p>Start with meter off (no test strip inserted).</p> <ul style="list-style-type: none">Press  to turn on the meter
2	 <p>Press and hold the time (for example 11:50) on the screen for 3 seconds until the screen changes.</p> 

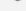Step	Action
3	 <p>12 h Time Format</p>  <p>24 h Time Format</p> <p>Set Time Format (12-hour or 24-hour clock)</p> <ul style="list-style-type: none">Press  or  to changePress  to continue <p>Note: The meter can display either a 12-h (1:23 PM) or 24-h (13:23) time format. If you prefer the 12-h format, there is no "AM". If setting a PM time, continue to press  until you see the "PM".</p>
4	 <p>+</p> <p>11:50 Hour</p> <p>Set Hour</p> <ul style="list-style-type: none">The hour blinks. Press  or  to set the hourPress  to continue
5	 <p>+</p> <p>11:50 Minute</p> <p>Set Minutes</p> <ul style="list-style-type: none">The minutes blink. Press  or  to set the minutesPress  to continue


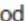

Set Date

Step	Action
6	 <p>Set Month</p> <ul style="list-style-type: none">• The month blinks. Press  or  to set the month• Press  to continue
7	 <p>Set Day</p> <ul style="list-style-type: none">• The day blinks. Press  or  to set the day• Press  to continue
8	 <p>Set Year</p> <ul style="list-style-type: none">• Press  or  to set the year• Press  to save. Setup complete

Enabling Out-of-Range Indicators and Quality Control Reminder







Introduction


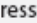
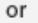






The out-of-range indicators and the quality control reminder can be enabled on the meter to support compliance with your facility's policies and procedures.

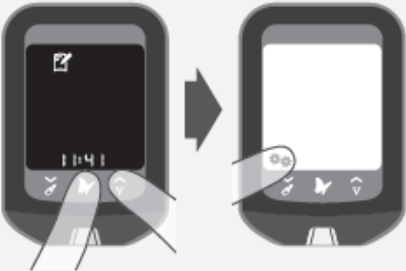- The out-of-range indicators notify the user with a  if the patient's blood glucose is lower than the set low-glucose threshold or a  if the patient's blood glucose is higher than the set high-glucose threshold.
- The quality control reminder will display  when a quality control test is due. Once the quality control test has been performed, this reminder will disappear until the next scheduled reminder. This reminder can be set for hourly (1-23 hours) or daily (1-30 days) intervals.

Note: Quality control reminder is for glucose control solution tests only. Performing a ketone control solution test will not clear the reminder.


In order to enable these features, use the following instructions.

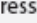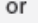Step	Action
1	 <p>Press  to turn meter on.</p>
2	 <p>Press and hold  and  at the same time for 4 seconds until the screen changes.</p> <ul style="list-style-type: none"> • Press and hold  until the screen changes to continue

Step	Action
3	 <p>Press  or  to set low out-of-range indicator level.</p> <ul style="list-style-type: none"> • Press  to continue <p>Note: --- will appear if the low out-of-range indicator is disabled.</p>
4	 <p>Press  or  to set high out-of-range indicator level.</p> <p>Note: --- will appear if the high out-of-range indicator is disabled.</p>
5	<p>Press  once to proceed to the quality control reminder setup or press  twice to exit setup.</p>



6

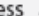


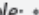
Press  to enable a quality control reminder.


Note: Leaving the --- will keep the quality control reminder disabled.

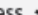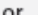

Press  or  to enable a quality control reminder by hour or by days.

Note: Hours are indicated as H (example: 12H is every 12 hours).

Days are indicated by the  icon (example:  is every 3 days).

- Press  to save your settings.



Preparing the Meter for Glucose and Ketone Testing

Calibrating the Meter for Glucose and Ketone Testing

Calibrate the meter to:	Calibration is required:
<ul style="list-style-type: none"> • Match the test strips being used 	<ul style="list-style-type: none"> • When using the meter for the first time
	<ul style="list-style-type: none"> • When using a new carton of test strips


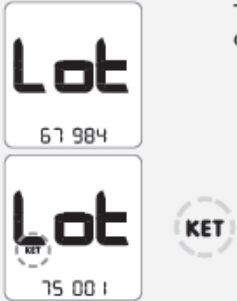


Remove the glucose calibrator (white) or the ketone calibrator (purple) from the new carton of test strips.

Peel the clear cover away from the corner with three bumps to open the calibrator package.

IMPORTANT: Use only the calibrator that is packaged in the carton of test strips you are using.

Preparing the Meter

Step	Action
1	 <p>Hold the calibrator with the LOT number facing you. Insert the calibrator into the meter until it stops. This turns the meter on.</p> <p>Notes:</p> <ul style="list-style-type: none"> Check that the meter screen is working properly each time you turn the meter on. If you see any white segments in the black off screen, or any black segments in the white test screen, there may be a problem with the meter. See "Using the Meter" section for more details.
2	 <p>The LOT number of the carton of test strips you are using appears on the display (example shown).</p>

Confirm the LOT Number

Check to see that the LOT number matches on all the following areas:

- Meter display
- Test strip calibrator
- Test strip instructions for use
- Test strip foil packet

IMPORTANT: Contact Customer Services if the LOT number does not match. **Do not** test your patient's blood glucose and blood ketone. The meter may produce inaccurate results.


LOT Number	What It Means	What To Do
<p>Glucose Match last five digits: XXXXX 67984 (example)</p> <p>Ketone Match first five digits: 75001 XXX (example)</p>	The meter is calibrated for the carton of test strips you are using.	You may now test your patient's blood glucose or blood ketone.
Does not match all	The meter may not be calibrated for the carton of test strips you are using.	Check to see that you are using the calibrator that came packaged in the carton of test strips you are using. Recalibrate the meter.

Remove the calibrator from the meter and store it in the carton of test strips.



Testing Blood Glucose or Blood Ketone




IMPORTANT: Only use a test strip once. Read the test strip instructions for use before performing a blood glucose or ketone test. It contains important information and will tell you how to store and handle the test strips.

Test Site



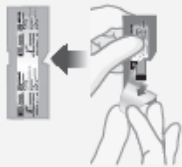

Step	Action
1	 <p>Fingertips</p> <p>Use fingertip blood samples for testing; see test strip instructions for use for additional blood sample types.</p>
2	Make sure that the sampling site is clean and dry before lancing to obtain a finger capillary blood sample. Check test strip expiry date. Do not use expired test strips; they may cause inaccurate results.

Performing a Blood Glucose or Blood Ketone Test

Step	Action
1	 <p>Open the foil test strip packet at the notch and tear down to remove the test strip.</p>
2	 <p>Insert the test strip into the meter until it stops. This will turn on the meter.</p> <p>Notes:</p> <ul style="list-style-type: none">• Check that the meter screen is working properly each time you turn the meter on. If you see any white segments in the black off screen, or any black segments in the white test screen, there may be a problem with the meter. See "Using the Meter" section for more details.• The meter turns off after 3 minutes of inactivity. Remove and reinsert the unused test strip to restart the meter.

Step	Action
<p>2 (cont.)</p>	<div data-bbox="257 132 442 487" data-label="Image"> </div> <p data-bbox="534 132 1173 221">Check that the LOT number of the test strip foil you are using appears. The  blinks, indicating the meter is ready for you to apply a sample to the test strip.</p> <p data-bbox="534 232 1160 291">Note: <i>KET</i> will appear on the screen if you have inserted a purple blood ketone strip.</p> <p data-bbox="534 360 1142 480">IMPORTANT: If testing glucose on venous blood, press the  to mark the test as a venous blood test until the  appears indicating that the meter is now ready for you to apply venous blood to the test strip.</p>
<p>3</p>	<div data-bbox="257 640 479 837" data-label="Image"> </div> <p data-bbox="534 629 1187 718">Obtain a blood sample. Collect the capillary blood using a single use lancing device and an appropriate technique.</p>

Step	Action
<p>4</p>	<div data-bbox="1600 138 1773 304" data-label="Image"> </div> <p data-bbox="1871 132 2542 221">Apply blood to the test strip. Bring the blood drop to the white area at the end of the test strip. The blood is drawn into the test strip.</p> <p data-bbox="1871 232 2516 291">Hold blood to test strip until you see 3 short lines on the meter screen. This means you have applied enough blood.</p> <p data-bbox="1871 304 1942 325">Notes:</p> <ul data-bbox="1871 339 2529 660" style="list-style-type: none"> • If you are testing blood glucose, you will see a 5-second countdown. If you are testing blood ketone, you will see a 10-second countdown. • Do not disturb or remove the test strip from the meter during the countdown. • If the countdown does not start, you may not have applied enough blood to the test strip. See test strip instructions for use for re-application instructions. If the countdown still does not start, remove the used strip and discard it properly. Start a new test with a new test strip. <div data-bbox="1600 373 1773 609" data-label="Image"> </div> <div data-bbox="1600 656 1773 891" data-label="Image"> </div>

Step	Action
5	<p data-bbox="518 137 1179 225">View the result. The test is complete when the result appears on the meter screen (examples shown). The result is stored in the memory.</p> <div data-bbox="268 137 414 291">  <p data-bbox="268 298 414 344">Blood Glucose Result example</p> </div> <div data-bbox="268 355 414 508">  <p data-bbox="268 515 414 562">Blood Ketone Result example</p> </div> <div data-bbox="268 572 414 726">  <p data-bbox="268 733 414 800">Venous Blood Glucose Result example</p> </div>
6	<p data-bbox="203 837 984 868">Press and hold  to turn off the meter. Discard the used test strip properly.</p>


Understanding Blood Glucose Test Results

The meter displays blood glucose results in mmol/L. The unit of measurement is preset. You cannot change this setting. The meter displays results from 1.1 - 27.8 mmol/L.


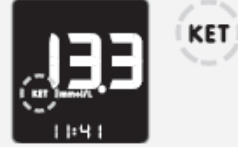
IMPORTANT:

- Low or high blood glucose results can indicate a potentially serious medical condition.
- If the blood glucose result appears to be inconsistent (lower or higher than expected), there may be a problem with the test strip. Repeat the test using a new test strip. Results that are incorrect may have serious medical consequences. Consult the prescribing physician before making any changes to diabetes medication plans.

Low Blood Glucose Results

If You See	What It Means	What To Do
 <p>A solid red arrow</p>	<p>If out-of-range indicators are enabled, appears when result is lower than the range set by your facility.</p>	<p>Follow your facility's procedure for handling out-of-range results.</p>
<p>LO</p> <p>Appears when result is lower than 1.1 mmol/L</p>	<p>Severe low blood glucose or There may be a problem with the test strip.</p>	<p>Repeat the test with a new test strip. This requires immediate action as defined by your facility's policies and procedures.</p>

High Blood Glucose Results

If You See	What It Means	What To Do
 <p>A solid yellow arrow</p>	<p>If out-of-range indicators are enabled, appears when result is higher than the range set by your facility.</p>	<p>Follow your facility's procedure for handling out-of-range results.</p>
 <p>KET</p>	<p>KET blinks three times and then disappears when: Blood glucose level is higher than or equal to 13.3 mmol/L.</p>	<p>Check blood ketone or follow your facility's policies and procedures.</p>
<p>HI</p> <p>Appears when result is higher than 27.8 mmol/L</p>	<p>Severe high blood glucose or There may be a problem with the test strip.</p>	<p>Repeat the test with a new test strip. This requires immediate action as defined by your facility's policies and procedures.</p>

Note: If you see the error message E-4, consult the "Error Messages" section in this Operator's Manual.

Understanding Blood Ketone Test Results

The meter displays ketone results in mmol/L, from 0.0 - 8.0 mmol/L. The unit of measurement is preset. You cannot change this setting.

Blood ketone is expected to be below 0.6 mmol/L.² High blood ketone may be caused by illness, fasting, vigorous exercise or uncontrolled blood glucose levels.¹⁻³

Repeat a blood ketone test using a new blood ketone test strip when:

- HI appears on the display
- The result is unusually high
- You question the result
- The blood ketone result is 0.0 mmol/L, but the blood glucose is higher than 16.7 mmol/L






Display	What It Means	What To Do
Result is between 0.6 – 1.5 mmol/L.	High blood ketone. A problem requiring medical assistance may be occurring.	Follow your facility's policies and procedures on how to handle elevated blood ketone results.



Display	What It Means	What To Do
Result is higher than 1.5 mmol/L.	The patient may be at risk of developing diabetic ketoacidosis (DKA). ²⁻⁶	Follow your facility's policies and procedures on how to handle high blood ketone results.
HI Appears when result is higher than 8.0 mmol/L.	Very high blood ketone or There may be a problem with the test strip.	Repeat the test with a new test strip. This requires immediate action as defined by your facility's policies and procedures.

View Logbook

The meter logbook can store up to 1,000 events – including blood glucose, blood ketone and control solution results, and other meter information.

Viewing Logbook Events



Step	Action
1	 While on the Home screen, press  to view logbook events.
2	<p>Examples: Press  or  to scroll through to view logbook events.</p>  <p>Blood Glucose Result</p> <p>Note: For today's results, only the time will be displayed at the bottom of the screen. For previous days results, the date will alternate with the time.</p>

	Venous Blood Glucose Result
	Control Solution Result

The  screen and control solution test result screen alternate.


Note: Check mark in the box indicates a control solution test and not that the result was within range.




Viewing Blood Glucose Averages

Step	Action
1	 While on the Home screen, press  to open the logbook.






Number of tests
in this average

While in the logbook, press  any time to view the 7-day average. [7-DAY]

- Press  to view the 14-day average. [14-DAY]
- Press  to view the 30-day average. [30-DAY]
- Press  to return to the logbook events.

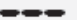Notes:

- *Averages do not include glucose control solution results.*
 - *Control solution results not marked as control solution tests may cause averages to be inaccurate.*
 -  appear on the meter screen when there are no current events or averages to view.
 - **LO** blood glucose test results are included as 1.1 mmol/L when calculating averages.
 - **HI** blood glucose test results are included as 27.8 mmol/L when calculating averages.
- Press  or  to scroll through 7-, 14- and 30-day averages.

WARNING: Averages should not be used if the meter is used on multiple patients. If the meter is used on multiple patients, the device cannot identify if data on the meter is from more than one patient.

Glucose and Ketone Control Solution Tests



A control solution test should be performed when:



- You are unsure of your patient's blood glucose or blood ketone results, or
- You receive a **QC** test reminder indicating that a quality control test is due.






Note: Once completed, **QC** will disappear.

IMPORTANT:

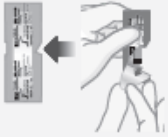
- Use only MediSense or Optium glucose and ketone control solutions with the meter.
- Control solution results should fall within the control solution range printed on the test strip instructions for use.
- Check that the LOT number printed on the test strip foil packet and instructions for use match.
- **Do not** use control solution past its expiry date. Discard control solution 3 months after opening or on the expiry date printed on the bottle, whichever comes first. (Example: open April 15, discard July 15; write the discard date on the side of the bottle.)
- The control solution range is a target range for control solution only, not for blood glucose levels.
- Replace the cap securely on the bottle immediately after use.
- **Do not** add water or other liquid to control solution.
- Contact Customer Services for information on how to obtain control solutions.

Performing a Control Solution Test

Step	Action
1	 <p>Open the foil test strip packet at the notch and tear down to remove the test strip.</p>
2	 <p>Insert the test strip into the meter until it stops. This will turn on the meter.</p> <p>Notes:</p> <ul style="list-style-type: none">• Check that the meter screen is working properly each time you turn the meter on. If you see any white segments in the black off screen, or any black segments in the white test screen, there may be a problem with the meter. See "Using the Meter" section for more details.• The meter turns off after 3 minutes of inactivity. Remove and reinsert the unused test strip to restart the meter.

Step	Action
3	 <p>The  blinks. Press the  until the  appears. The meter is now ready for you to apply control solution to the test strip.</p> <p>Note: KET will appear on the screen if you have inserted a purple blood ketone strip.</p>
4	 <p>Apply control solution to the test strip. Shake the control solution bottle to mix the solution. Apply a drop of control solution to the white area at the end of the test strip in the area shown. The control solution is drawn into the test strip.</p>

5

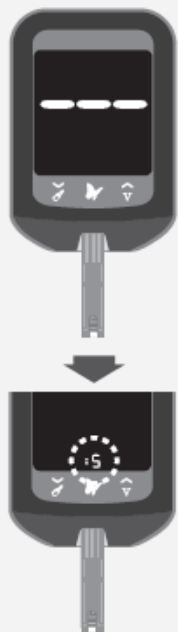

Hold the control solution to the test strip until:

- You see 3 short lines on the meter screen. This means you have applied enough control solution and the meter is reading the control solution.

Notes:

- If you are testing with a blood glucose strip, you will see a 5-second countdown. If you are testing with a blood ketone strip, you will see a 10-second countdown.
- Do not** disturb or remove the test strip from the meter during the countdown.
- If the countdown does not start, remove and discard the used test strip, turn off the meter and try again with a new strip.

6



Glucose Control
Solution Result
example



Ketone Control
Solution Result
example

View the result.

The test is complete (examples shown) when the result appears on the meter screen. The result is stored in the memory as a control solution result.

Compare the control solution result to the range printed on the blood glucose or blood ketone test strip instructions for use. The result should fall within the range.

Note: KET appears with the result if performing a ketone control solution test.

Out-of-Range Control Solution Results:

- Repeat the test if control solution results are outside the range printed on the test strip instructions for use.
- Stop using the meter if control solution results are consistently outside the range printed on the test strip instructions for use. Contact Customer Services.

Transferring Meter Data to a Computer

Transferring meter data to a computer requires a compatible data management system. You will also need a micro USB cable to connect the meter's USB port to a computer.

For more information, contact your Sales Representative or Customer Services.

WARNING: To avoid the possibility of electric shock, never perform a test while the meter is connected to the computer.

Error Messages

Message	What It Means	What To Do
E-1	The temperature is too hot or too cold for the meter to work properly	<ol style="list-style-type: none">1. Move the meter and test strips to a location where the temperature is within the test strip operating range. (See test strip instructions for use for the appropriate range.)2. Wait for the meter and test strips to adjust to the new temperature.3. Repeat the test using a new test strip.4. If the error reappears, contact Customer Services.
E-2	Meter error	<ol style="list-style-type: none">1. Turn off the meter.2. Repeat the test using a new test strip.3. If the error reappears, contact Customer Services.

Message	What It Means	What To Do
E-3	Blood drop is too small or Incorrect test procedure or There may be a problem with the test strip	<ol style="list-style-type: none"> 1. Review the testing instructions. 2. Repeat the test using a new test strip. 3. If the error reappears, contact Customer Services.
E-4	The blood glucose level may be too high to be read by the system or There may be a problem with the test strip	<ol style="list-style-type: none"> 1. Repeat the test using a new test strip. 2. If the error reappears, contact the prescribing physician Immediately.
E-5	Blood was applied to the test strip too soon	<ol style="list-style-type: none"> 1. Review the testing instructions. 2. Repeat the test using a new test strip. 3. If the error reappears, contact Customer Services.

Message	What It Means	What To Do
E-6	Calibration error or Test strip error	<ol style="list-style-type: none"> 1. Check the date setting on the meter. 2. Check the expiry date on the test strip foil packet. 3. Repeat the calibration using the calibrator that came with the test strip you are using. 4. If the error reappears, contact Customer Services.
E-7	Test strip may be damaged, used or the meter does not recognise it	<ol style="list-style-type: none"> 1. Check that you are using the correct test strip for this meter. (See test strip instructions for use to verify the strip is compatible with this meter.) 2. Repeat the test using a test strip for use with the meter. 3. If the error reappears, contact Customer Services.
E-9	Meter error	<ol style="list-style-type: none"> 1. Turn off the meter. 2. Repeat the test using a new test strip. 3. If the error reappears, contact Customer Services.

Maintenance



Replacing the Batteries



You will see this screen when the batteries are low.

Notes:

- Meter settings and logbook information will be saved when you change the batteries.
- The time and date will need to be set again if the batteries are not replaced within 1 minute.

Step	Action
1	 <p>Turn meter over and slide open the battery door on the side as shown.</p>
2	 <p>Remove the old batteries.</p>

Step	Action
3	 <p>Install new batteries with (+) facing up. Note: The meter uses 2 replaceable CR 2032 coin cell batteries.</p>
4	 <p>Slide the door back into place until it clicks. Note: The next time you turn the meter on, it may prompt you to reset the time and date. See "Setting Up the Meter" section for more details.</p>

Note: When you no longer need the meter, remove the batteries and dispose of batteries and meter in compliance with your local regulations.



The European Battery Directive requires separate collection of spent batteries, aiming to facilitate recycling and to protect the environment. The batteries in this product should be removed and disposed in accordance with local regulations for separate collection of spent batteries.

Cleaning the Meter

Step	Action
1	Assemble cleaning supplies: <ul style="list-style-type: none">• Cloth dampened with:<ul style="list-style-type: none">• Mild detergent/soap and water, or• 70% isopropyl alcohol, or• A mixture of 1 part household bleach, 9 parts water
2	<p>IMPORTANT: Do not place the meter in water or other liquids. Avoid getting dust, dirt, blood, control solution, water or any other substance in the meter's test strip port, USB port and battery compartment.</p> <p>Gently clean the exterior of the meter.</p>
3	Allow the meter to dry.

Troubleshooting

Problem	What It Means	What To Do
Test strip is inserted in the strip port and nothing happens.	Test strip is not inserted properly or fully into the meter	1. With the contact bars (3 black lines) facing up, insert the test strip into the meter until it stops. This turns on the meter. 2. If the meter still does not turn on, contact Customer Services.
	No batteries are installed; Batteries installed incorrectly	See "Maintenance" section on how to properly install batteries.
	Dead batteries	Change batteries. Reset date and time, if needed.
	Meter may be plugged into a computer (PC appears on meter screen)	Unplug the meter from the computer.
	Problem with the test strip	Try a new test strip.
	Problem with the meter	Contact Customer Services.

Problem	What It Means	What To Do
The test does not start after applying the blood sample.	Blood sample is too small	<ol style="list-style-type: none"> 1. See test strip instructions for use for re-application instructions. 2. Repeat the test using a new test strip. 3. If the test still does not start, contact Customer Services.
	Sample applied after meter turns off	<ol style="list-style-type: none"> 1. Review the testing instructions. 2. Repeat the test using a new test strip. 3. If the test still does not start, contact Customer Services.
	Problem with meter or test strip	<ol style="list-style-type: none"> 1. Repeat the test using a new test strip. 2. If the test still does not start, contact Customer Services.

Meter Specifications	
Assay method	Amperometry
Automatic shutoff	At least two minutes of inactivity
Battery life	Up to 3,000 tests
Measurement range	For blood glucose testing 1.1 - 27.8 mmol/L For blood ketone testing 0.0 - 8.0 mmol/L
Memory	Up to 1,000 events, including blood glucose, blood ketone and control solution results, and other meter information
Minimum computer requirements	System must only be used with EN60950-1 rated computers. Use a USB certified cable
Operating relative humidity	10% to 90% (non-condensing)
Operating temperature	Meter: 10 °C to 50 °C (50 °F to 122 °F) System: See test strip instructions for use

Power source	Two CR 2032 lithium (coin cell) batteries
Size	5.97 cm (w) x 8.68 cm (l) x 0.87 cm (d) 2.35 in (w) x 3.42 in (l) x 0.34 in (d)
Storage temperature	-20 °C to 60 °C (-4 °F to 140 °F)
Altitude	See test strip instructions for use
Weight	33 g to 37 g (1.2 oz. to 1.3 oz.) including batteries

Note: For test strip specifications, see test strip instructions for use.

Electromagnetic Compatibility (EMC): FreeStyle Optium Neo H meter has been tested for both electrostatic discharge and radio frequency interference. Emissions are low and unlikely to interfere with other nearby electronic equipment. To limit radio frequency interference do not use the FreeStyle Optium Neo H meter near cellular or cordless telephones, radio transmitters or other electrical or electronic equipment that are sources of electromagnetic radiation, as these may interfere with the proper operation of the meter. Avoid use of the device in very dry environments, as electrostatic discharges from synthetic materials (e.g., carpets) could cause damage.

References

- Schade DS, Eaton RP. Metabolic and clinical significance of ketosis. *Special Topics in Endocrinology and Metabolism* 1982; 4:1–27.
- Wiggam MI, O’Kane MJ, Harper R, Atkinson AB, Hadden DR, Trimble ER, Bell PM. Treatment of diabetic ketoacidosis using normalization of blood 3-hydroxybutyrate concentration as the endpoint of emergency management. *Diabetes Care* 1997; 20:1347–1352.
- Harano Y, Kosugi K, Hyosu T, Suzuki M, Hidaka H, Kashiwagi A, Uno S, Shigeta Y. Ketone bodies as markers for Type 1 (insulin-dependent) diabetes and their value in the monitoring of diabetes control. *Diabetologia* 1984; 26:343–348.
- Ubukata E. Diurnal variation of blood ketone bodies in insulin-dependent diabetes mellitus and non-insulin-dependent diabetes mellitus patients: The relationship to serum C-peptide immunoreactivity and free insulin. *Ann Nutr Metab* 1990; 34:333–342.
- Luzi L, Barrett EJ, Groop LC, Ferrannini E, DeFronzo RA. Metabolic effects of low-dose insulin therapy on glucose metabolism in diabetic ketoacidosis. *Diabetes* 1988; 37:1470–1477.
- Hale PJ, Crase J, Natrass M. Metabolic effects of bicarbonate in the treatment of diabetic ketoacidosis. *Br Med J* 1984; 289; 1035–1038.

Warranty Statement

Limited Warranty

Abbott Diabetes Care Inc. ("Abbott") warrants that the Abbott Diabetes Care Blood Glucose Monitor ("Meter") shall be free from defects in material and workmanship for a period of two (2) years from the original date of purchase or delivery date, whichever is later, provided it is not modified, altered or misused. Under this Limited Warranty, if the Meter is defective in material or workmanship, Abbott's sole obligation is to replace the Meter, free of charge, with a same or similar meter as determined by Abbott in its sole discretion.

Abbott warrants the performance of the Meter in accordance with its specifications if it is used as directed and provided that the failure to perform or misperformance of the Meter has not been caused in whole or in part by the use of test strips that are not the test strips manufactured by Abbott.

This Limited Warranty covers only the Meter and shall not apply to auxiliary equipment or disposable accessories, extends only to the original purchaser, and is not assignable or transferable.

Please see the User's Manual for Instructions for Use, distributor and manufacturer information.

For warranty service, contact Customer Services for assistance and/or instructions for obtaining a replacement Meter. See User's Manual for your Customer Services contact information. Abbott may require as a condition to obtaining warranty service that you return your Meter, postage prepaid, to an address specified by Customer Services.

TO THE EXTENT POSSIBLE UNDER LAW, THESE ARE ABBOTT'S COMPLETE EXPRESS WARRANTIES FOR THE METER AND STATE YOUR EXCLUSIVE REMEDIES, ABBOTT MAKES NO OTHER WARRANTIES, EXPRESS OR IMPLIED, AND ABBOTT EXCLUDES AND DISCLAIMS ANY OTHER WARRANTIES INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE, AND NONINFRINGEMENT AND ANY OTHER REMEDIES INCLUDING, BUT NOT LIMITED, TO COMPENSATIONS UNDER CONTRACT OR TORT LAWS. ABBOTT DOES NOT WARRANT THAT OPERATION OF THE METER WILL BE UNINTERRUPTED OR ERROR FREE.

Limitation of Liability

TO THE EXTENT POSSIBLE UNDER LAW, ABBOTT WILL NOT BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES RESULTING DIRECTLY OR INDIRECTLY FROM USE OF THE METER OR FAILURE OF THE METER TO PERFORM IN ACCORDANCE WITH SPECIFICATIONS.

Some jurisdictions do not allow the exclusion or limitation of other express or implied warranties or incidental or consequential damages, so the above limitations or exclusions may not apply to you.

Distributed by:

Abbott Diabetes Care
666 Doncaster Road
Doncaster 3108
Victoria, Australia
ABN 95000 180 389
1800-801-478

AS MEFO
Jarveotsa tee 50c
Tallinn 13520
Estonia
3726707076

Abbott Laboratories Ireland, Ltd
Abbott Diabetes Care
Block B
Liffey Valley Office Campus
Quarryvale, Dublin 22
D22 X0Y3
Ireland
1800 77 66 33

SIA FARMEKO
Brivibas str. 410
LV-1024 Riga
Latvia
800 03081 (Local Toll Free)

UAB "Abovita"
Žemaites str. 21
LT-03118 Vilnius
Lithuania
8 800 20055
8 5 233 71 41

Abbott Laboratories (N. Z.) Ltd,
Ground Floor Bldg D,
4 Pacific Rise,
Mount Wellington,
P.O. Box 22-801 Otahuhu,
Auckland, New Zealand
0800 106 100

Abbott Diabetes Care
Mediserv
P.O.Box 40758
Riyadh 11511
Saudi Arabia
800 124 1238

Abbott Laboratories (Singapore)
Pte Ltd
Abbott Diabetes Care
3 Fraser Street #23-28
DUO Tower
Singapore 189352
1800-272-2881

Abbott Laboratories
Abbott Place
219 Golf Club Terrace
Constantia Kloof
1709
Johannesburg
South Africa
0800 222 688
+27 11 858 2375

Abbott Laboratories Ltd
Abbott Diabetes Care
Abbott House
Vanwall Business Park
Vanwall Road
Maidenhead
Berkshire
SL6 4UD
UK
0800 170 1177

Уповноважений представник
і Дистриб'ютор .
ПП «НВФ» VD MAIS»
Відділ Медичної Техніки
Вул. М.Донця,б,
03061, Київ, Україна
Тел. +380 44 499 29 89


Authorized Representative
Abbott Laboratories
(Malaysia) Sdn Bhd
Abbott Diabetes Care
No. 22, Jalan Pemaju U1/15,
Seksyen U1
HICOM-Glenmarie
Industrial Park
40150 Shah Alam,
Selangor, Malaysia
03-5566-3388

FreeStyle and related brand marks are trademarks of Abbott Diabetes Care Inc in various jurisdictions

Patent <https://www.abbott.com/patents>



Abbott B V
Wegalaan 9, 2132 JD Hoofddorp,
The Netherlands

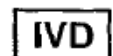

Abbott Diabetes Care Ltd
Range Road
Witney, Oxon
OX29 0YL, UK